

HENROPLATE®



USER MANUAL

HENROPLATE®

December 2023

Table of content

Table of content	2
Important information.....	3
Intended use	3
Storage	3
Instruction for use	4
Precautions	4
Daily operation	4
Warnings	5
Environmentally friendly disposal	5
Technical datasheet HenroPlate®	6

Important information

Before using HenroPlate® read these instructions closely.

Intended use

HenroPlate® are designed and intended for food only.

HenroPlate® has been developed by Danish occupational therapists to ensure a high-quality PLATE for people that experience loss of strength and coordination. HenroPlate® Is BPA Free and can safely be used by children under the age of 3 years.

HenroPlate® has a high back and a low front, which gives a sloped design. The sloped design allows the food to slide down to one side making eating with one hand possible.

HenroPlate® is delivered with small rubber feet, that can be attached on the bottom, this way HenroPlate® will stand firmly on the table

HenroPlate® can with advantage be used in hospitals, nursing homes, residences and special institutions.

Storage

Store dry, at room temperature and out of direct sunlight.

Instruction for use

Content: 1 HenroPlate® comes with 4 sets of feet.

1. Unpack the plate and feet, dispose of the packing properly.
2. Before use wash the plate.
3. After wash: break off 4 feet and place them in the 4 hole on the bottom of the plate.
4. The HenroPlate® are now ready for use.

Precautions

Always wash HenroPlate® before use.

HenroPlate® can be used for all foods.

Do not heat HenroPlate® over a candle light or fire.

Daily operation

HenroPlate® can be washed in Dish Washer with up to 65° C degrees and industry dish washer up to 80° C for up to 3 minutes.

Remove the rubber feet when washing.

The HenroPlate® can handle Microwave.

HenroPlate® can handle temperature range between -20° C to 100° C.

Warnings



REMOVE THE FEET BEFORE WASH

Environmentally friendly disposal

HenroPlate® has a PET mark on the bottom with material information. Please dispose the HenroPlate® in an environmentally friendly neutral way.



HenroPlate® are made of material which should be recycled. Please therefore dispose the old HenroPlate® via suitable collection systems.



IF YOU HAVE ANY DOUBT OF WHEN TO DISPOSE THE HENROPLATE®, YOU CAN SEND AN EMAIL TO HENROTEK@HENROTEK.COM

Technical datasheet HenroPlate®	
Material	PP - Polypropylene
Marked	CE
Weight	202 G
Height	28,5 cm
Width	19,5 cm
Dishwasher	80° C degrees
Freezer	-20° C degrees
Microwave	Yes
BASIC UDI-DI No.	57048315C4
UDI-DI	05704831129026
Standards and Regulations	Regulation (EC) 1935/2004 of 27. October 2004 Regulation (EC) 2023/2006 of 22 December 2006 Danish executive order, BEK nr 681 of 25/05/2020 Regulation (EC) 10/2011 of 14 January 2011 Regulation (EU) 2017/745 of 5 April 2017
Accessories Material	HenroPlate® Feet (TPE – Thermoplastic Elastomer)